



Market: Professional
Healthcare and Consumer
Healthcare

Composition: Acetone

Product Approvals: FDA
registered Class 1 device.

Shelf life: 3 years from production
date.

Product number and Packaging:
Standard sizes are:

- MD0013-P - 16oz (473 ml)
- MD0013-Q - 32oz (946 ml)



Mavidon
6625 White Dr
Riviera Beach, FL 33407 USA
www.mavidon.com
561-585-2227

ACETONE

REMOVAL OF COLLODION

The Acetone from Mavidon removes **collodion** from electrode sites.

Acetone penetrate tapes and soften adhesives to facilitate removal.

The product may be used for all age groups

Intended Use:

Collodion: Apply liberally to each electrode site. After removing all the electrodes, comb out as much of the collodion residue as possible. If there is any residue reapply acetone with a saturated gauze until residue is remove.

However, please take caution because acetone will melt LDPE plastics (hearing aids, glasses, car seats, phones....etc)

Use with care:

- **Acetone** is extremely flammable
- May discolor or remove furniture paints.
- May melt low density plastics.
- May dissolve exam gloves.
- For External Use Only!
- May Cause Eye Irritation.
- **Do not use on open wounds!**
- Keep Container tightly closed.
- Discontinue if skin irritation occurs.

