

# COLLODION II



**MEDICAL APPLICATION:** Used to attach electrodes to the skin or scalp. Primarily, but not limited, for EEG monitoring (Epilepsy, Sleep Disorders, Ambulatory...). Especially useful when it is important to maintain a secure contact for monitoring. When applied correctly, collodion will hold even when the skin is hairy, perspiring, or oily.

Clean surface to be bonded with LemonPrep®, PEDIAPREP™ to remove the body oils and dead skin. Apply a thin layer of Collodion to the electrode or gauze. To facilitate cure, force clean cool air over the electrode with an air pump to aid solvent evaporation

**SPECIAL EFFECTS:** The easiest way to make scars and some burn effects is with Collodion.

**Market:** Professional Healthcare and Consumer Healthcare, Consumers and Makeup Artist.

**Composition:** Solvent-based, non-plasticized, cellulose nitrate solution with very fast curing; non-brittle with 10% elongation. Ingredients: Acetate, ethanol, nitrocellulose, colloid, granular.

**Product Approvals:** FDA registered Class 1 device.

**Shelf life:** 18 months from production date.

## USE WITH CARE:

- Extremely flammable
- Use in well ventilated areas or with correct air purifier.

**SHIPPING REGULATIONS:** Hazardous - Flammable Liquid. SEE MSDS!

## Product number and Packaging:

Availability	Description
MDCOLII-L	32 oz (946 ML)
MDCOLII-P	16 oz (473 ml)
MDCOLII-2	2 oz (60 ml) 6 PACK



Mavidon  
6625 White Dr  
Riviera Beach, FL 33407 USA  
[www.mavidon.com](http://www.mavidon.com)  
561-585-2227